 *Bedienungsanleitung*
 *Instruction Manual*
 *Manual de instrucciones*
 *Mode d'emploi*

ECO – MATIC[®] *mit Flaschenaufsatz*
with bottle attachment
con adaptador para frascos
avec porte-bouteille

by **HSW**

for veterinary use only

2 ml or 5 ml

Henke-Sass, Wolf GmbH
Kronenstrasse 16
D-78532 Tuttlingen
Germany
Tel: +49 74 61 189 209
Fax: +49 74 61 189 208
www.henkesasswolf.de
E-mail: HSW@henkesasswolf.de

700038 – 02/05

ECO – MATIC®
by **HSW**

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
ECO – MATIC®
by **HSW**

- Automatic self-filling syringe for injections
- with bottle attachment

- A) ergonomic designed handle**
- B) plastic barrel with bottle attachment**
- C) durable plastic basket**
- D) metal needle fitting, Luer Lock**
- E) dosage setting**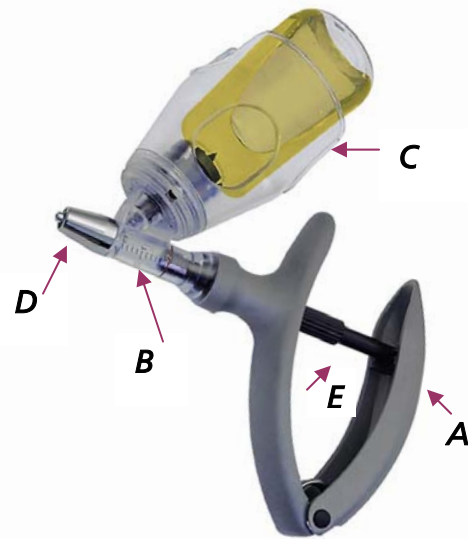

Instruction for use 2 ml or 5 ml

The ECO – MATIC® self-filling dosing syringe is designed for continuous use, only the O-ring and the barrel can be changed if necessary.

It is essential that the syringe is cleaned before each use. Please follow the manufacturers recommendations with any cleaning or disinfectant solutions used.

1) TO FIT THE NEEDLE

- **Attention:**
For your safety: Leave the protective cover (only on disposale needles) on the needle until ready for use
- insert needle onto Luer Lock fitting and secure the needle clockwise
- disconnect the needle fitting the opposite way

2) CONNECT THE BOTTLE

- pierce the rubber cap of the bottle with a sterile needle before you attach the bottle into the spike
- insert the bottle into the basket
- line up the bottle, by using the viewing window of the basket, in the centre of the rubber cap and pierce the spike into the bottle while rotating the bottle slightly


3) EXCHANGE THE BOTTLE

- pull out the bottle by using the grip hollow on both sides of the bottle

4) SET DOSE


- Turn the dose adjuster either clockwise or counter – clockwise until front of the piston is aligned with the dose setting printed on the barrel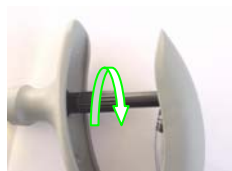

Dose setting

5) FILLING OF THE ECO – MATIC®

- a) remove protection cover of the needle
- b) hold the ECO – MATIC® at a slight incline and take care that the spike is below the liquid to avoid suck in of air into the barrel
- c) spueeze adjustable handle and then release quickly to draw fluid into the barrel
- d) to remove air bubble inside the barrel, repeat step (c)

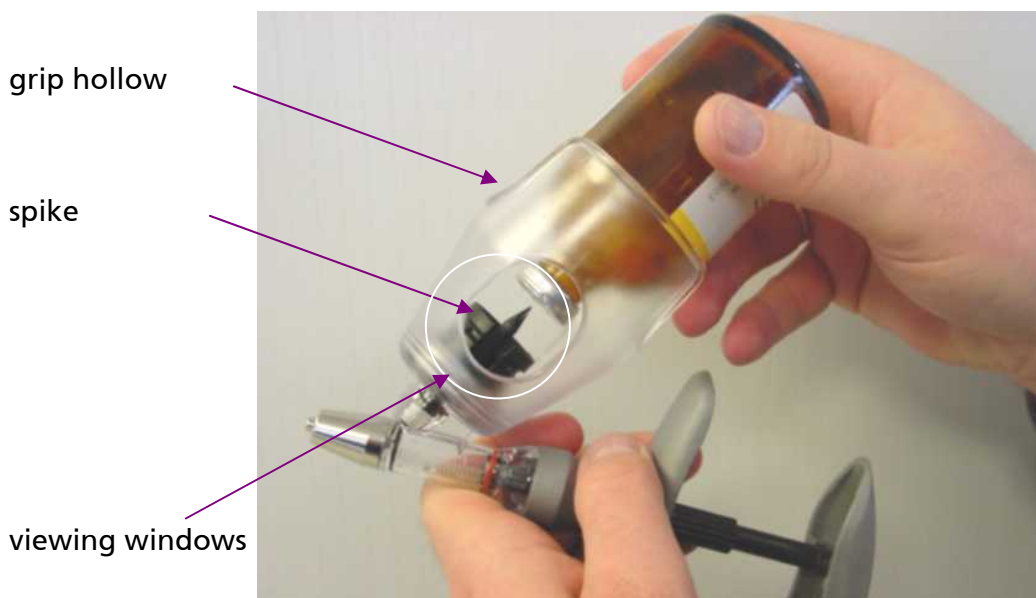INSTRUCTION AFTER USE OF THE ECO - MATIC®

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix
Rinse thoroughly with clean, warm water.

Disinfection

It is recommended that the ECO – MATIC® and a supply of needles or nozzles be disinfected before and after each use.

- 1) Suck "hot" water into the barrel by using an attached bottle with warm water
- 2) Suspend the complete ECO – MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes
- 3) Remove the ECO – MATIC® from container, wrap cloth around the handle and pump it dry

TO LUBRICATE:

If piston becomes sluggish: Remove piston and lubricate piston O-ring with olive oil or Vegetable Oil. Do not use mineral oil, silicone, WD-40, glycerin or alcohol. Petroleum based products harden the o-rings, reducing the life of the component.

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